



Kangaroo 924 Pump Directions

Beginning Administration

1. Press **ON**.
2. Wait while pump performs system check. (Displays "BBBB" for a moment.)
3. ***Press **VOL**, then **VOL CLR** daily, to clear out volume delivered.***
4. Rate of delivery may be changed when the delivery rate is shown on the display by using the adjustment knob or by pressing **▲** or **▼** button. Depressing and holding button will cause numbers to continue to increase or decrease.

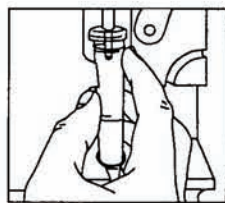


Figure 4A

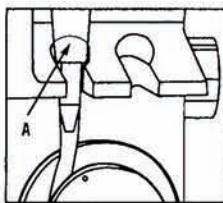


Figure 4B

5. Slide drip chamber into upper drip chamber guide (See Figure 4A).
6. Seat and secure chamber in lower drip chamber guide (A).
7. Remove distal connector protective cover from the feeding set..
8. Follow KANGAROO 924 Feeding Set instructions to prepare for feeding.
9. Slowly open control clamp and fill entire line with fluid, leaving a few drops of fluid in the drip chamber.
NOTE: AVOID FILLING THE DRIP CHAMBER MORE THAN HALF FULL. Close control clamp.
10. Attach distal connector to feeding tube.
11. Inspect lens in OPTIX lock area (B). Clean any contaminants from lens before proceeding to the next step (Figure 5B).

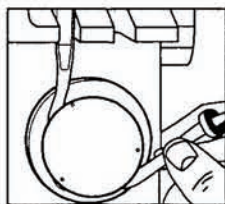


Figure 5A

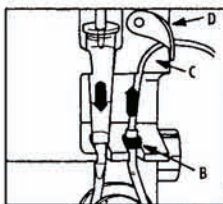


Figure 5B

12. Grasp silicone tubing and gently stretch around rollers (See Figure 5A). Insert OPTIX connector into OPTIX lock (B). Thread tubing over tubing guide (C). Secure latch over tubing (D) (See Figure 5B). Do not over-stretch silicone tube.

VERIFY THAT THE FEEDING SET IS PROPERLY LOADED IN THE PUMP BEFORE PROCEEDING. AN IMPROPERLY LOADED PUMP SET COULD RESULT IN UNCONTROLLED FLOW AND POSSIBLE PATIENT INJURY.

13. Do NOT set VTBD
14. Open control clamp.
15. Press **START/HOLD**.
16. Proceed with feeding.
17. When feed complete or after 24 hour use, dispose of pump sets (Set not to exceed 24 hours of use).