

# ACCLAIM Pump



IV Pharmacy & Nutrition Services

455 Reservation Road, Marina CA 93933

Phone: 831.384.8080 Fax: 831.384.8065

1. Spike and Prime tubing manually
2. Follow directions for loading a set (see picture, or directions on side of pump`)
3. Connect end of tubing to catheter of patient
4. Press yellow **ON/OFF** button
5. Press PRIMARY Blue **VTBI** button and enter desired volume
6. Press PRIMARY Blue **RATE** button and enter desired rate  
*If you wish to program in TIME in minutes instead, PRESS VTBI again, or Until the word "time" appears. Enter desired time in minutes*
7. Unclamp clamps on tubing and patients catheter
8. Press Green **START** button

## Notes:

To clear the Total Volume delivered press the large Blue CLR VOL button

To clear any individual value entered; press the small Blue CLR button

For "AIR IN LINE" message, Press and hold down the Purge gray button (*under White DOSE CALC button*). Press and hold until air is visually past air sensor

To reset program without changes:

1. Press yellow **ON/OFF** button
2. Press big blue **CLR VOL** button
3. Press PRIMARY blue **VTBI** button and enter desired volume
4. Press green **START** button